# newDuet<sup>®</sup>

ENGLISH

PP 3-13

FRANÇAIS

PP 15-25

ITALIANO

PP 27-37

ESPAÑOL

PP 39-49

DEUTSCH

PP 51-61

**OPERATING INSTRUCTIONS  
AND  
MAINTENANCE MANUAL**

**S-SCORT® newDuet® Model 2314**

**AC SUCTION UNIT WITH BATTERY BACK-UP  
100V-240V AC POWER**

**30 LPM PERFORMANCE  
DC BATTERY BACK-UP**

**CONTROLLED BATTERY  
MAINTENANCE**

**SSCOR, Inc.**  
11064 Randall St.  
Sun Valley, CA 91352  
USA

Tel Int. +1 818 504 4054  
Fax +1 818 504 6032  
USA 800 434 5211

[www.sscor.com](http://www.sscor.com)  
email: [marketing@sscor.com](mailto:marketing@sscor.com)

# Table of Contents

S-SCORT® newDuet® Model 2314

Caution-Notice	4
General Description and Symbols	5
Getting Acquainted- Basic Instructions	6 – 7
General Specifications-Trouble Shooting	8
Warranty-Maintenance	9
Internal Access-Internal Components	10
Retention Bracket Mounting Instructions	11
Declaration of Conformity	12

## Caution-Notice

S-SCORT® newDuet® Model 2314

1. Federal law restricts this device to sale, distribution, and use by, or on the order of a physician, emergency medical technician, or other medical practitioner. For use by medical personnel trained in suctioning techniques and in the use of medical suction equipment.
2. This manual is restricted to the discussion of the use and maintenance of this device. It does not attempt to discuss professional techniques in suctioning procedures.
3. Operator should be thoroughly familiar with these operating instructions before this device is used.
4. Use a thumb control on all suction tips, especially when suctioning children and intubated patients. The regulator's low setting can be used when suctioning children and intubated patients.
5. Reconnect to AC charging source after each use. Check battery condition indicator lights daily
6. The battery is protected from a deep discharge condition by shutting down after approximately 45 minutes ( $\pm 10\%$ ) running time. Reconnect to AC power to restart.
7. Do not use in the presence of flammable agents or anesthetics.
8. The shut-off valve in the canister lid will close down when the canister is full.
9. Install a new canister before testing for vacuum over 300mmHg to minimize the possibility of implosion, which can occur when a canister is aged or damaged.
10. Where the integrity of the external protective earth conductor arrangement (ground) is in doubt, the unit shall be operated from its internal electrical power source (battery only). Grounding reliability can only be achieved when connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".
11. To discern the condition of the battery when the unit is disconnected from the power source turn the unit on and observe the battery condition LED 's on the control panel.

S-SCORT® newDuet® Model 2314  
AC suction unit with battery back-up  
US Patent Pending

ETL listed, conforms to UL STD 2601-1, IEC 60601-1, and EN 60601-1-2, Certified to CAN/CSA STD C22.2 NO 601.1  
©2000 - 2003 SSCOR, Inc., S-SCORT, Duet, HI-D and Big Stick are registered trademarks of SSCOR, Inc.

# General Description



S-SCORT® newDuet® Model 2314

S-SCORT newDuet® is a portable, 100V-240V AC constant suction device, with a 12V DC battery back-up. The battery is charged by an internal, DC Dual Mode battery charger. A fully charged battery will power the unit for 45 minutes (+/-10%). Battery condition is automatically monitored and visualized by indicator lights on the control panel. AC charging should bring the battery to a dependable working charge in 6 to 8 hours. Suction power can be regulated when full power may be considered harmful to the patient.

**CAUTION: S-SCORT suction units are designed for hospital crash cart suction, patient transport and emergency medical service. They are not designed or intended for use in extended procedures that require prolonged high vacuum/low airflow applications, as is the case in wound drainage or endoscopic use or in any other procedure that produces high vacuum levels within an occluded system for an extended period of time. Turn the S-SCORT suction unit off when it is not in use.**

## Description of Symbols

S-SCORT® newDuet® Model 2314

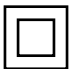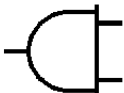SYMBOL	LOCATION	MEANING
	Control panel	Pump is on
	Serialized label	Type B equipment
	Control panel	Battery level of charge
		Green LED = Full
		Green/Yellow LED = High
		Yellow LED = Half
		Yellow/Red LED = Low
	Control panel	Red LED = Battery depleted Put back on Charge
	Control panel	Push On / Push Off
100-240 V 	Serialized label	Alternating Current (100-240 V)
	Serialized label	Class II Equipment / Double Insulation
	Control panel	Power source connected

# Getting Acquainted

S-SCORT® newDuet® Model 2314

## Running on AC Power

1. Connect the male plug on the AC power cord (1) to a grounded wall receptacle.
  - 1A. Connect the female plug to the facing side of the unit (1A).
2. Check the control panel (2). The “power source connected” orange LED (3) shows a good connection and battery is charging.

## Test for vacuum

3. Attach patient tubing to patient port (12) on collection canister (13).
4. Press the ON/OFF switch (5) on the control panel to start the vacuum pump.
5. Occlude patient tube and turn vacuum regulator (7) clockwise to the stop.
6. Observe gauge (8). The vacuum reading should rise to 300mmHg from the zero setting in 2 seconds. It should max out at approximately 525 mmHg.

**NOTE 1:** If the vacuum does not reach 525mmHg, see Trouble shooting, page 8. Refer internal vacuum problems to qualified personnel.

**NOTE 2:** After the vacuum test, readjust the vacuum reading to your desired level.

## Running on DC Power

While the unit is still running, disconnect the AC power cord (1) to test the function of the battery.

1. Check battery condition indicators (4), see page 5.

**SPECIAL NOTICE:** A blinking LED on the control panel indicates low battery capacity. This means the battery has been subjected to irreparable damage (see page 9). **REPLACE BATTERY!**

Keep the newDuet® connected to AC power at all times, except for portable use. The battery will not overcharge.

**Weekly Inspection:** Establish a weekly checklist to assure performance of the equipment.

1. Confirm the AC cord is supplying AC power. Check power indicator lights on the Control Panel (3).
2. Run the unit.
3. Check for vacuum, occlude patient tubing.
4. Check the regulator (7), adjust to desired setting

# General Specifications

S-SCORT® newDuet® Model 2314

CHARACTERISTICS	SPECIFICATIONS
Size	17”L x 9”H x 6”W
Weight	8.6 lbs.
Vacuum Pump	12V DC oil-less diaphragm, 3.0A Exceeds 525 mmHg Clinical Airflow = 30LPM
Variable Regulator (7)	Controls vacuum from 50 to 525+ mmHg
Gauge (8)	Calibrated in mmHg. Color-coded
Power Source: Battery (DC Power)	Rechargeable Sealed Lead Acid 45 (±10%) minute running time
Medical switcher (AC Power)	100V - 240VAC, 47 - 63Hz
Switch (5)	Uses 1ea. 2A 250V fast acting fuse Off/Standby/On membrane switch
AC Receptacle (1A)	Uses (2) 2.5A 250 V fast acting fuses
Collection Canister (13)	1200cc SSCOR part #48041
Patient tubing	Vinyl tubing 9/32”ID, 72”L SSCOR part #43200
Suction Tip	HI-D® . “Big Stick®”. SSCOR part #44241

## Trouble Shooting

S-SCORT® newDuet® Model 2314

PROBLEM	POSSIBLE CAUSE	CORRECTIVE ACTION
Does not function when switch is on (DC Power)	Battery discharged	Reconnect to AC power to activate charger
Blinking LED	Low battery capacity	Replace battery
Power source connected	Fuse blown	Replace fuse (see above) indicator light not lit
	AC power cord disconnected (1-1A)	Reconnect cord
No suction when pump is running	Vacuum line (10) loose	Tighten
	Canister (13) defective	Replace
	Canister lid (13) loose	Tighten
	Catheter thumb vent is open	Occlude
	Debris has collected in pump	Refer to maintenance section of this manual

# Warranty

S-SCORT® newDuet® Model 2314

SSCOR warrants that each new product is free from defects in material and workmanship under normal use and service for a period of one year from date of purchase.

If returned to SSCOR, we will arrange for repairs or replacement within the terms of the warranty. The defective product should be decontaminated and returned properly packaged and postage prepaid. Loss or damage in transit to the factory shall be at the purchaser's risk.

Please call 800.434.5211 - Int. +1 818.504.4054 for return authorization. Loss or damage in return shipment from SSCOR shall be at the purchaser's risk.

The warranty shall not apply to any SSCOR product which has been repaired by anyone other than an authorized SSCOR representative, or altered in any way so as, in SSCOR's judgment, to affect its safety or efficacy, nor which has been subject to misuse, negligence, or accident, nor which has had the serial number altered, effaced or removed.

Neither shall this warranty apply to any SSCOR product which has been connected otherwise than in accordance with the instructions furnished by SSCOR.

This warranty is in lieu of all other warranties expressed or implied and of all other obligations or liabilities on SSCOR's part, and SSCOR neither assumes, nor authorizes any representative or other persons to assume for it, any other liability in connection with the sale of SSCOR products.

Batteries and disposable collection canisters are excluded from this warranty.

## Model 2314 Battery Replacement Policy

SSCOR will replace any S-SCORT newDUET® Model 2314 battery which fails to operate the pump to specifications for a period of three years from the date of purchase. Verify the condition of the battery per the weekly checklist on Page 7 in this Operations Manual.

# Maintenance

S-SCORT® newDuet® Model 2314


- The unit is designed to suction contaminated fluids, which should be removed from the system immediately after the procedure. Dispose of the collection canister, patient tube and catheter and replace. If the procedure produced an excessive quantity of fluids, check the vacuum line (10) for evidence of moisture. Replace vacuum line if moisture is evident. (Use 3/8" ID tubing.)
- If the vacuum line between the pump and canister is moist, it is possible that fluids have reached the vacuum pump. Your engineering department will have to open the unit to check the condition of the pump.
- The capacity of the battery should be checked periodically by running the unit on battery only. Detach from AC power and run the unit for 15 minutes and look for a blinking LED (page 7).
- If the Medical Switcher Power Supply (F) appears defective, return the unit to the factory for repair. Do not attempt to repair the Medical Switcher Power Supply.
- If the Internal Dual Mode Charger (E) appears defective, return the unit to the factory for repair. Do not attempt to repair the Internal Dual Mode Charger.
- Cleaning: Disconnect AC power source prior to cleaning. Use a mild detergent. Do not use alcohol, solvents or cleaning solutions. For interior cleaning, use dry, compressed air.
- **For technical assistance, call (800) 434.5211, Int. +818.504.4054**

**Internal Access for Electrical and Pneumatics**

S-SCORT® newDuet® Model 2314

**WARNING: DO NOT ATTEMPT TO SERVICE THIS EQUIPMENT IF YOU ARE NOT A BIOMEDICAL TECHNICIAN**

To Remove Canister Holder: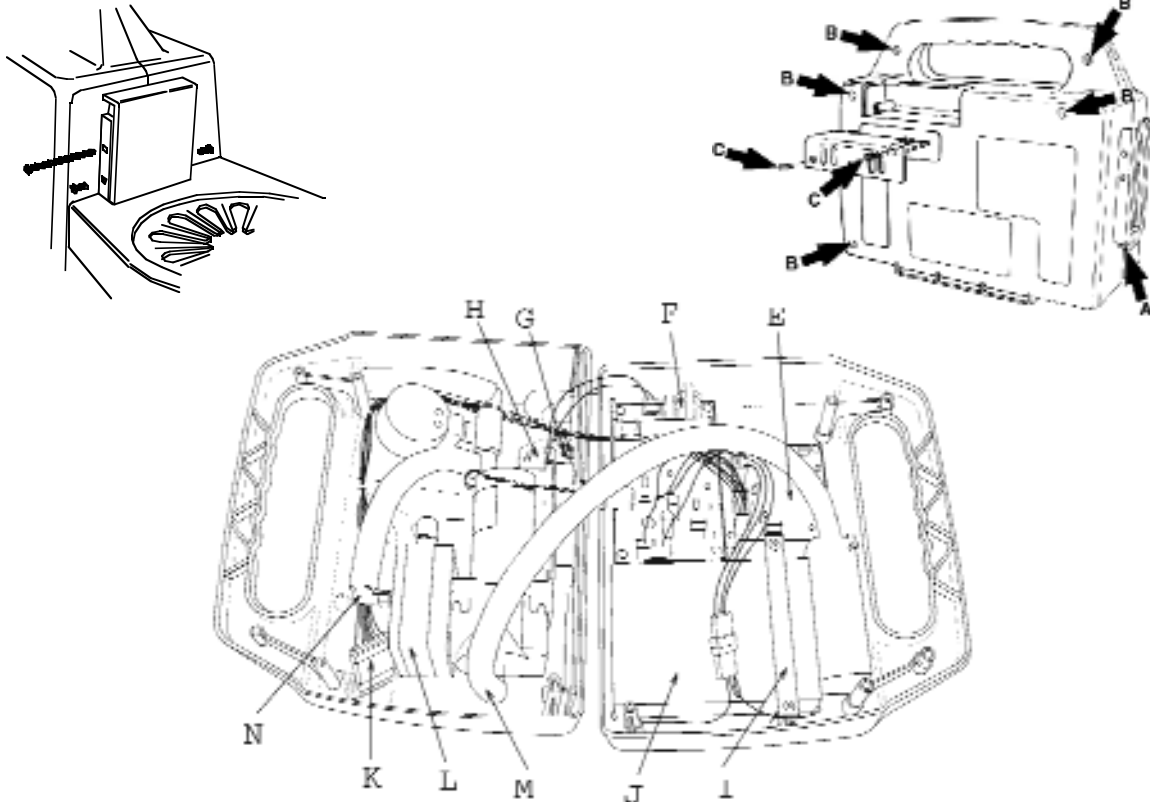

**TO REMOVE CANISTER HOLDER:**

Remove one 4” screw and two set screws on the canister bracket. Lift canister holder up and out of the main chassis.

**OPEN/ CLOSE**


Clamshell design enables access to components. All 6- 32 phillips head fastening screws located in back of the unit.

**REMOVE MAIN CHASSIS SCREWS**

- (A) 1 screw under the canister holder.
- (B) 2 screws in the back of the handle.
- (B) 3 screws in the back of the main chassis.
- (C) It should not be necessary to open the exhaust door unless fluids have entered the system. Slowly pull the unit apart at the handle to observe internal components.
- (E) **PC BOARD- ELECTRICAL CIRCUITS**  
Do not attempt to service the PC Board. Return to SSCOR or authorized service center for service.
- (F) Power Supply (Medical Switcher). Do not attempt to service the Power Supply. Return to SSCOR or authorized service center for service for service.
- (G) Ground is connected between the receptacle and the power supply.
- (H) AC receptacle, Hospital Grade. Two 250V- 2A fuses are in the receptacle fuse drawer.
- (I) Battery Bracket.
- (J) Battery, 12V, Sealed Lead Acid.
- (K) Wiring connection to the control panel.
- (L) Vacuum pump. Return to SSCOR or authorized service center for service.
- (M) Exhaust barb.
- (N) Vacuum barb.

# Retention Bracket Mounting Instruction

S-SCORT® newDuet® Model 2314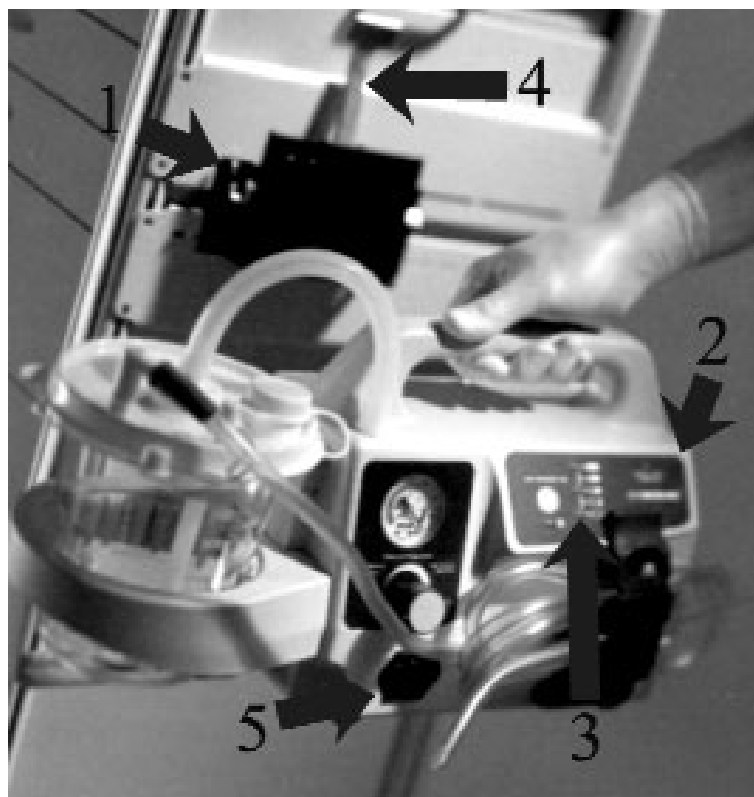

Mount the retention bracket using the four holes on the side of the shelf (1). Mount the retention bracket so the Model 2314 suction unit can be released from the bracket easily and there is enough room to work the handle (4). The control panel (2) should be easily viewed and the battery condition LED's (3) visible when the unit (5) is connected to a grounded hospital receptacle. When placing the suction unit into the retention bracket, be sure the unit snaps into the bracket and is securely held in place.

For further instructions, please see the operations manual.

# DECLARATION OF CONFORMITY


*SSCOR, Incorporated  
11064 Randall Street  
Sun Valley, CA 91352  
United States of America  
Tel: (818) 504-4054  
Fax: (818) 504-6032*

**Type of Equipment:** AC Powered & Portable Aspiration Pump  
**Model Name:** S-SCORT new-Duet®  
**Model Type Number:** 2314, 2314B,

**We herewith declare that the above mentioned product meets with the following requirements of the EC Directive concerning Medical Devices – 93/42/EEC**

**Safety:** EN 60601-1: 1990  
With A1 & A12:1993, A2:1995 & A13:1996  
**EMC:** EN 60601-1-2: 1993  
**Classification** Class IIa  
**Rules of Classification** Annex IX, Rules 6 & 11  
**Annex used for CE Declaration** Annex VI & Annex VII  
**Notified Body** SEMKO, AB

**Signature, Date:**

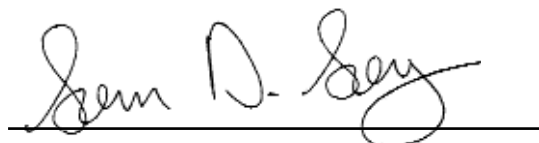**Name:**  
**Title:**

Sam D. Say  
President

**Authorized European Representative**

Dynamed UK Ltd  
Unit 3 Stag Business Park  
Deer Park Court  
Donnington Wood  
Shropshire TF2 7NA, UK  
01952 56 56 56 Phone  
01952 56 56 57 Fax

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**Gemeldete Behörde** SEMKO, AB